# BegoStone plus

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# BegoStone plus

Stone of high hardness  
and low expansion

## Instructions for use

BegoStone plus is a stone of the class 4, which is characterized by its exceptional hardness, its resistance to abrasion, its extremely smooth surface and its very low expansion of 0.09 %. BegoStone plus with its physical properties it is far above the requirements of DIN EN ISO 6873.

### Physical properties

Colour	ivory 35
Soaking time	20 seconds
Pouring time at approx. 20 °C	approx. 5 minutes
Setting start (Vicat test)	approx. 10 minutes
Resistance to pressure after 1 hour	60 MPa
Setting expansion	0.09 %
Hardening time	approx. 30 minutes

### Use

Inlay, crowns and bridges as well as partial denture technique

### Mixing proportions

100 g BegoStone plus + 20 ml water or  
1 measuring spoon BegoStone plus  
(approx. 30 g) +  
1 graduation of measuring cup water  
(approx. 6 ml)

### Preparation

Pour BegoStone plus into the water.  
Wait 20 seconds and then spatulate a short time  
on the vibrator.

### Mixing time

1. All dental duplicating materials as well as silicone duplicating materials such as Wirozil®:  
mechanically (BEGO vacuum mixing unit)  
approx. 45 seconds.  
Fill in the mould on the vibrator.
2. Reversible duplicating materials such as Wirogel® M:  
mechanically (BEGO vacuum mixing unit)  
approx. 60 seconds,  
manually approx. 120 seconds.  
Fill in the mould on the vibrator.

### Availability

1 tin = 4,5 kg  
1 tub = 18 kg

**REF 56045**  
**REF 56046**

### Attention

Protect from humidity, close tin/tub after use.

### Warranty

Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values.

Our products are subject to a constant further development.  
Therefore alterations in construction and composition are reserved.

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